

Getting Started

Welcome to the Diagnostics training series! This guide walks you through accessing your account and introduces the Diagnostics homepage where you'll manage your patient records.

[Go to Diagnostics](#)

Accessing Your Diagnocat Account


Use the Recommended Browser

For the best experience, open **Google Chrome** before navigating to Diagnocat.

Log In Steps

1. Go to diagnocat.com
2. Select your region
3. Click **Login**
4. Choose your Diagnocat product subscription to enter the correct platform version
5. Enter your email and password
6. Click **Login** to access your account

Need help? Use the "**Forgot Password**" link on the login page.

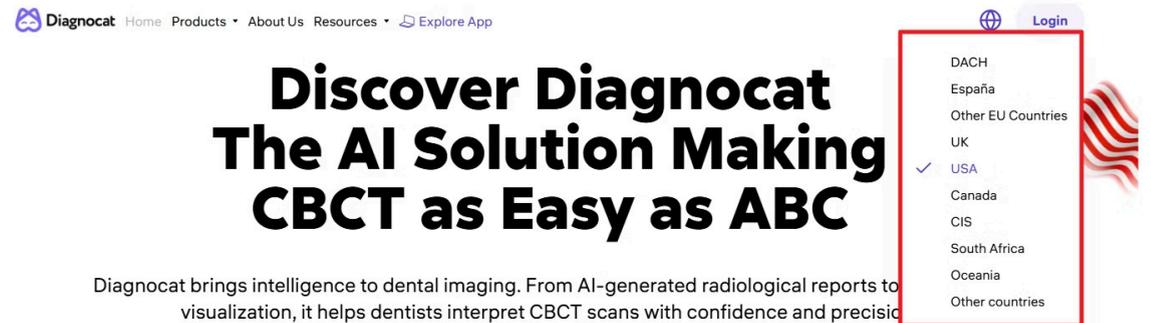


Diagnocat Home Products About Us Resources Explore App

Discover Diagnocat The AI Solution Making CBCT as Easy as ABC

Diagnocat brings intelligence to dental imaging. From AI-generated radiological reports to visualization, it helps dentists interpret CBCT scans with confidence and precision.

- DACH
- España
- Other EU Countries
- UK
- ✓ USA
- Canada
- CIS
- South Africa
- Oceania
- Other countries

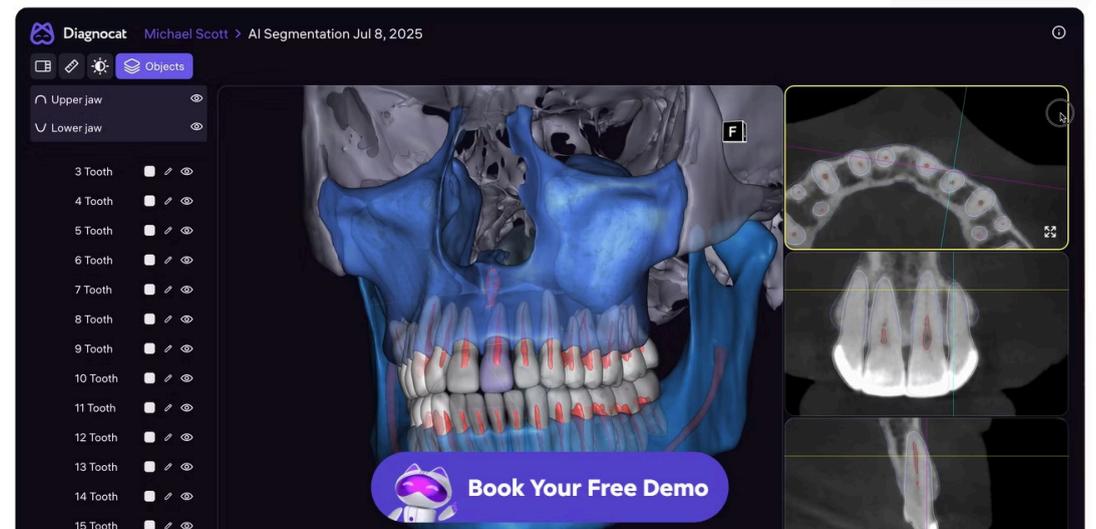


Diagnocat Michael Scott AI Segmentation Jul 8, 2025

Upper jaw
Lower jaw

- 3 Tooth
- 4 Tooth
- 5 Tooth
- 6 Tooth
- 7 Tooth
- 8 Tooth
- 9 Tooth
- 10 Tooth
- 11 Tooth
- 12 Tooth
- 13 Tooth
- 14 Tooth
- 15 Tooth

Book Your Free Demo

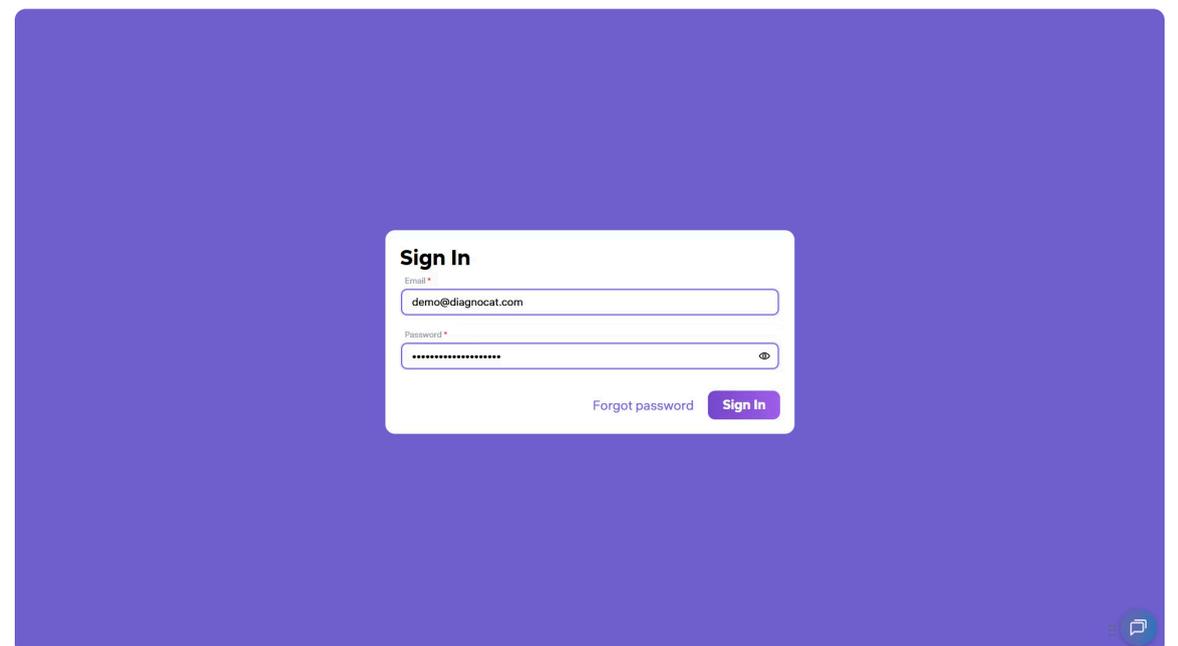

Sign In

Email *

demo@diagnocat.com


Password *

[Forgot password](#) [Sign In](#)

STEP 2


Navigating the Homepage

Once Diagnocat is connected to your image viewing software, your homepage becomes your **central hub** for patient management.

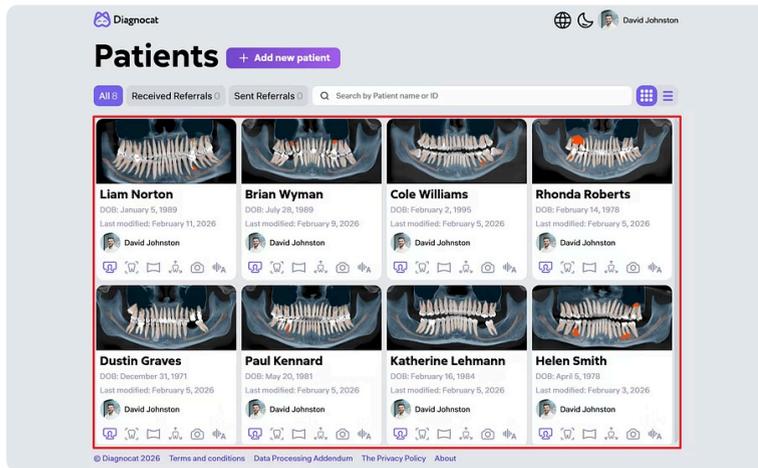

Patient List

Located in the center of the screen, providing quick access to all patient profiles.

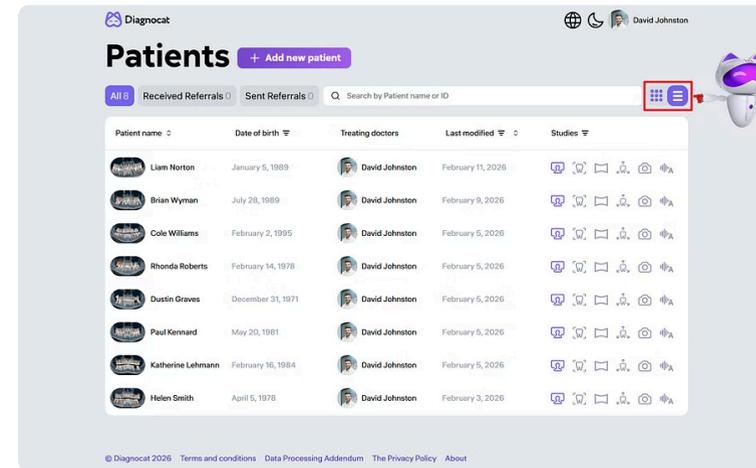

Switch Views

Toggle between **list view** and **matrix view** depending on your preference.

Search & Interface Controls

Use the **search bar** at the top of the page to find patients by name or ID. Diagnocat will instantly display matching profiles.

In the upper right corner, you'll find a set of helpful buttons:



Knowledge Base

Opens guides, videos, and updates



Language

Selects the interface language



Theme Toggle

Switches between dark and light modes

Diagnocat

Patients

+ Add new patient


All 8 Received Referrals 0 Sent Referrals 0 Search by Patient name or ID

<p>Liam Norton DOB: January 5, 1989 Last modified: February 11, 2026 David Johnston</p>	<p>Brian Wyman DOB: July 28, 1989 Last modified: February 9, 2026 David Johnston</p>	<p>Cole Williams DOB: February 2, 1995 Last modified: February 5, 2026 David Johnston</p>	<p>Rhonda Roberts DOB: February 14, 1978 Last modified: February 5, 2026 David Johnston</p>
<p>Dustin Graves DOB: December 31, 1971 Last modified: February 5, 2026 David Johnston</p>	<p>Paul Kennard DOB: May 20, 1981 Last modified: February 5, 2026 David Johnston</p>	<p>Katherine Lehmann DOB: February 16, 1984 Last modified: February 5, 2026 David Johnston</p>	<p>Helen Smith DOB: April 5, 1978 Last modified: February 3, 2026 David Johnston</p>

© Diagnocat 2026 Terms and conditions Data Processing Addendum The Privacy Policy About

Creating a New Patient Profile

Click **Add Patient** to begin creating a new profile.

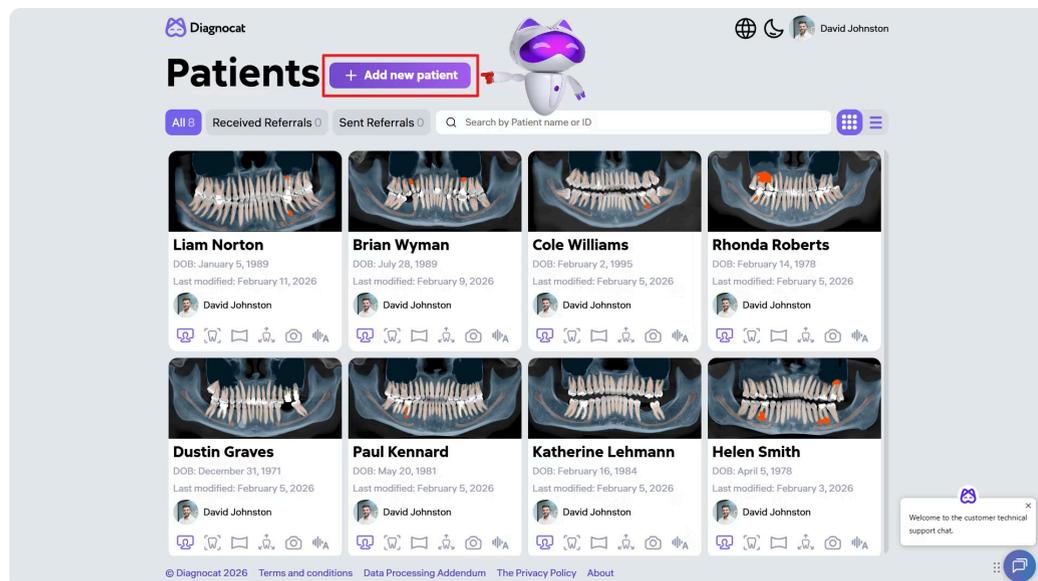

Enter Patient Information

1. First name and last name
2. Email (optional)
3. Select date of birth
4. Assign a treating doctor from your clinic
5. Review the ID field — add a unique patient identifier (e.g., clinic ID, medical record number)
6. Click **Add**

The new patient will **immediately appear** in your patient list.

The screenshot shows the 'New patient' form overlay on the Patients page. The form is titled 'New patient' and contains the following fields:

- First Name ***: Paul
- Last Name ***: Norton
- Email**: _____@gmail.com
- Date of birth ***: 01-28-1970
- Patient ID**: 0001
- Gender ***: Male (selected), Female, Other
- Treating doctor ***: David Johnston (selected)

At the bottom of the form, there are two buttons: 'Cancel' and 'Add'. The 'Add' button is highlighted with a red box.

Organizing Your Patient List

Sorting helps you quickly locate the information you need. You can sort patients by the following criteria:



Name



Date of Birth



Study Type



Last Modified



Attending Doctor

Diagnocat David Johnston

Patients + Add new patient

All 8 Received Referrals 0 Sent Referrals 0

Patient name	Date of birth	Treating doctors	Last modified	Studies
Dustin Graves	<input checked="" type="checkbox"/> Adults <input type="checkbox"/> Children Clear Apply	Select... <input type="checkbox"/> David Johnston <input type="checkbox"/> Katherine Smith Clear Apply	<input type="radio"/> All <input type="radio"/> Today <input type="radio"/> One Week <input checked="" type="radio"/> Custom From: MM-DD-YYYY <input type="text"/> To: MM-DD-YYYY <input type="text"/> Clear Apply	<input checked="" type="checkbox"/> CBCT <input type="checkbox"/> IOXRay <input type="checkbox"/> Pano <input type="checkbox"/> Intraoral Scan Clear Apply
Paul Kennard	January 5, 1989	David Johnston	February 3, 2026	
Katherine Lehmann	February 14, 1978	David Johnston	February 5, 2026	
Liam Norton	April 5, 1978	David Johnston	February 9, 2026	
Rhonda Roberts	February 2, 1995	David Johnston		
Helen Smith	July 28, 1989	David Johnston		
Cole Williams				
Brian Wyman				

© Diagnocat 2026 [Terms and conditions](#) [Data Processing Addendum](#) [The Privacy Policy](#) [About](#)

Continue to the next training guide to explore more Diagnocat features.

Launch Diagnocat